

SOLAX MOBILE ARMREST – Model M-AM01-13

Installation Instructions

1.

Locate the armrest bar underneath the seat. Then insert the end of the armrest bar as shown by the arrow.



2.

Secure the armrest as shown using the hardware that is included .



3.

Repeat steps 1 and 2 for the other side of the armrest.



How to Fold and Unfold Armrests

FOLDING THE ARMREST

<p>Original Position</p>  A front view of a wheelchair with its seat and backrest in an upright position. The armrests are extended outwards from the seat.	<p>Folded Position</p>  A front view of the wheelchair with the seat and backrest folded down. The armrests are pushed inward and folded down towards the seat.	<p>How to Fold Armrest</p>  A front view of the wheelchair with numbered arrows indicating the steps: 1. Upward arrow on the backrest, 2. Inward arrow on the armrest, 1. Upward arrow on the armrest, 2. Inward arrow on the backrest.	<ol style="list-style-type: none">1. Fold the back part of the seat down.2. Lift the armrest up3. Then push the armrests inward to fold down
---	---	---	--

UNFOLDING THE ARMREST

<p>1.</p>  A front view of the wheelchair with the backrest folded down and the armrests pushed outward. Arrows point outwards from the armrests. <p>Push out the armrest and lock into place</p>	<p>2.</p>  A front view of the wheelchair with the backrest folded down and the armrests in place. An upward arrow points to the backrest. <p>Lift the rear part of the seat up</p>
---	---